

SINGLE USE FACE MASKS




**chuck howard associates inc. Canada**



519-660-3619 Email: [chuckhow@rogers.com](mailto:chuckhow@rogers.com) [howardassociateslighting.com](http://howardassociateslighting.com)





Size of mask(Unexpanded) : 9.5cm x 17cm

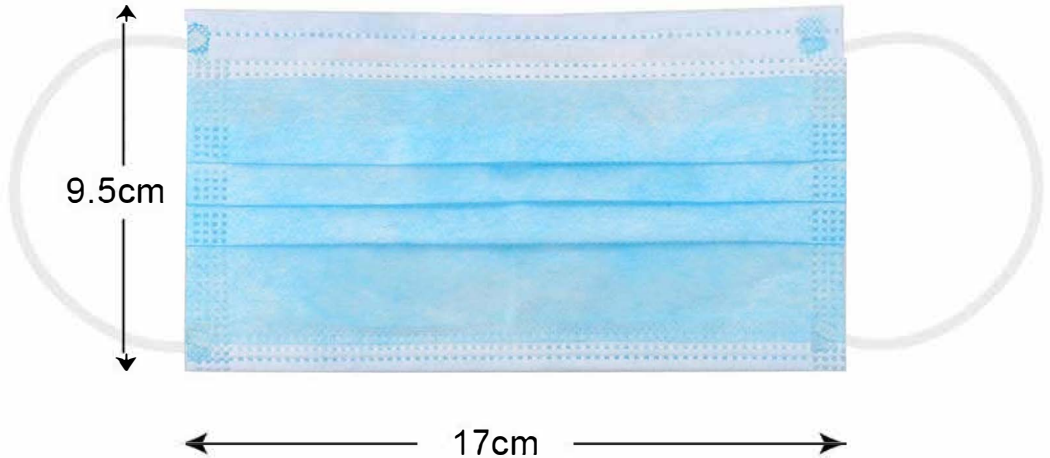

**chuck howard associates inc. Canada**



519-660-3619 Email: [chuckhow@rogers.com](mailto:chuckhow@rogers.com) [howardassociateslighting.com](http://howardassociateslighting.com)

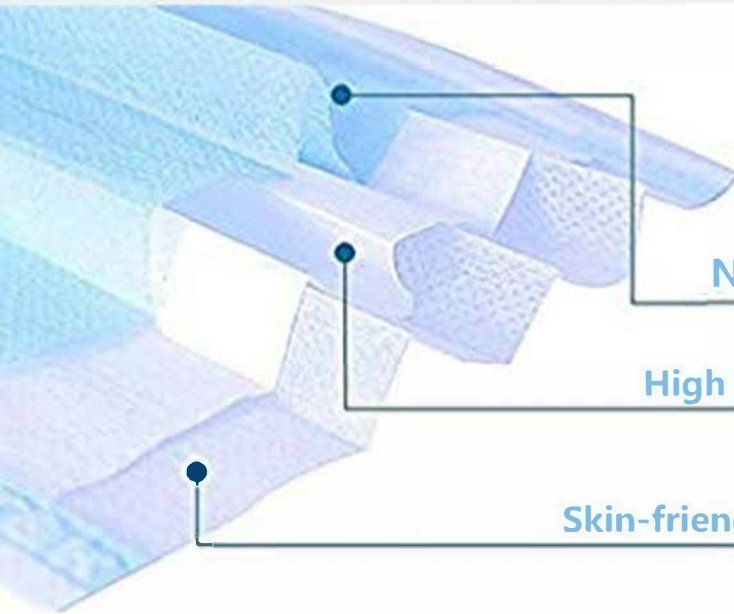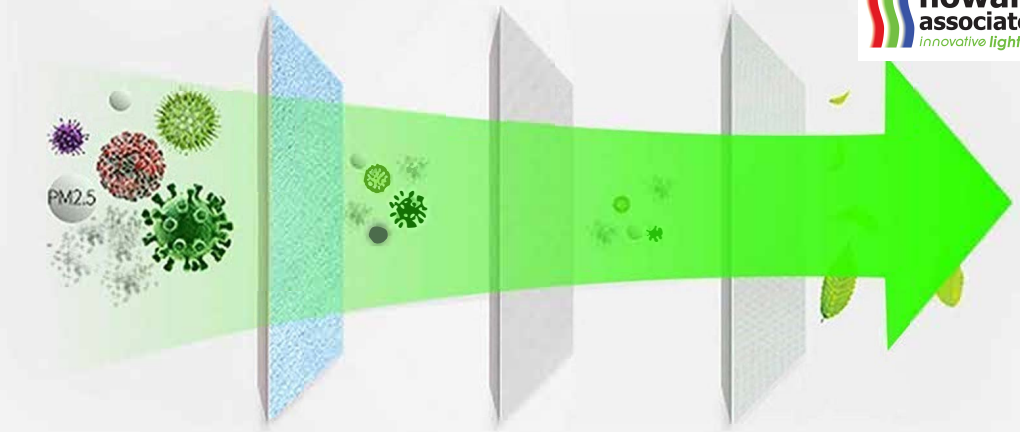

## 3 Layers

Non-woven fabric

High Density Filter Layer

Skin-friendly Composite Fiber

# Adjustable nose bridge clip



Built-in nose bridge clip .Adjustable camber  
to fit the face more



## Mask wearing steps



1

Hold the mask with the nose clip side up and stick the mask to the face



2

Gently press the nose clip to fit the bridge of the nose



3

Press the nose clip and pull the lower end of the mask to the jaw, and ensure that it is tight



4

Gently press around so that the mask fits the face as a whole, comfortable to wear

**chuck howard associates inc. Canada**



519-660-3619 Email: [chuckhow@rogers.com](mailto:chuckhow@rogers.com) [howardassociateslighting.com](http://howardassociateslighting.com)